horseAlarm™

SAFETY FOR YOUR HORSE

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Copyright LUDA Elektronik AB

This guide is published by LUDA Elektronik AB.

All rights reserved. Any rights not expressly granted herein are reserved.

© LUDA Elektronik, 2011

EN

ES

FI

IT

PL

SV

English

User Guide

Contents

- 1. Introduction5
- 2. Safety precautions / Important information6
- 3. What is included in your horseAlarm package7
- 4. Explanation of the included products8
- 5. horseAlarm – How It Works.....10
- 6. Before you begin the installation.....11
- 7. Understanding and testing before use.....12
- 8. Placement of harness, transmitter & repeater.....16
- 9. Mounting the antenna17
- 10. Receiver mode – Easy.....18
- 11. Battery symbols18
- 12. Alarm modes19
- 13. Receiver mode – Advanced20
- 14. Receiver mode – Settings20
- 15. If you have more than one horse.....21
- 16. Frequently Asked Questions21
- 17. Troubleshooting.....22
- 18. Warranty23
- 19. Information regarding recycling23
- 20. Contact us, accessories and support23

1. Introduction

Congratulations to your purchase of horseAlarm™ from LUDA Elektronik AB!

Your horseAlarm-solution will make foaling safer and more secure. The portable and wireless receiver will allow you to supervise the wellbeing of your horse(s) from the comfort of your home.

We hope that the horseAlarm-solution will satisfy your needs and will exceed your expectations.

The product is CE-certified which means that it fulfils all established rules and demands for usage within the EU. The CE-certification further implies that the radiation level is below the established radiation levels stated in the EMC directive 89/336/EEC and R&TTE directive (1999/5/EC).

We thank you for purchasing the horseAlarm-solution and we hope you'll easily get through the installation process. If questions arise during the installation, please read through the different troubleshooting chapters and the FAQ chapter. If further questions arise you are very welcome to contact our friendly customer support.

Before you start reading about the installation of your horseAlarm-solution we ask you to read through chapter 2. Safety precautions / Important information.

For news on our product range or accessories to your horseAlarm-solution, please visit our website: www.horsealarm-europe.com

LUDA Elektronik AB

EN

ES

FI

IT

PL

SV

2. Safety precautions / Important information

**Always use the correct power adaptor for each product.
The products will be damaged if they are supplied with wrong voltage!**

Do not use the products in the following places:


Do not use the products in hospitals since they might interfere with sensitive equipment.

Do not use the products on-board airplanes since they might interfere with sensitive equipment.

- Always follow the laws and legislations that apply in your country.
- Do not leave plastic packaging material near children to avoid the risk of suffocation.
- Do not place any of the included power adaptors outside or close to water.
- Do not cover or place the power adaptors on heat sensitive materials.
- Do not lift the receiver in the antenna.
- Do not use several Transmitters with the same frequency within the same area since they will interfere with each other.

LUDA Elektronik AB resigms all responsibility that might occur from abuse or incorrect usage of the products. Always follow the instructions in this User Guide. LUDA Elektronik AB cannot guarantee that horseAlarm always will alarm when foaling, the product should be seen as an aid during the foaling period.

3. Included in your horseAlarm-package:

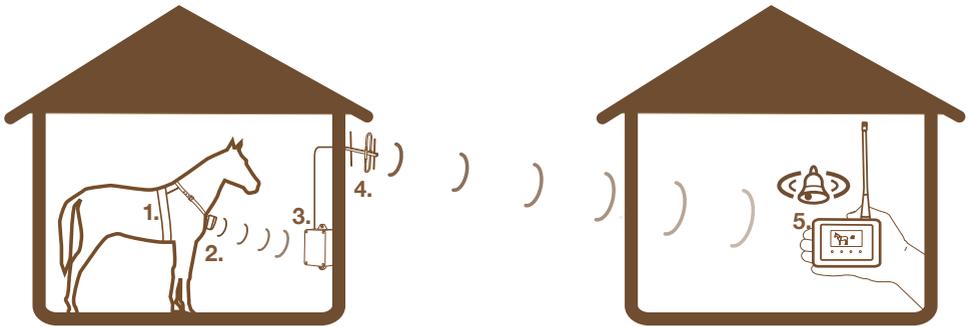EN

ES

FI

IT

PL

SV

Your horseAlarm-solution includes the following products:

1. 1 Harness with built in sensors. (Art. nr 656SHA)
2. 1 Transmitter.
3. 1 Repeater, for retransmission of the signal. (Art. nr 653RLH)
4. 1 Transmitting R300-antenna. (Art. nr 350AHA)
5. 1 Wireless receiver with LCD-monitor, history function and alarm. (Art. nr 651RXH)

Your horseAlarm-solution also includes:

- 1 12V power adaptor for the Repeater.
- 1 12V power adaptor for the Receiver.
- 3 AAA batteries for the Transmitter.
- 1 7,5 m antenna cable between the Repeater and the R300-antenna.
- 2 pcs of shrink tube for weather protection of antenna cable joints.
- 3 Cable ties
- 1 Surcingle with rubber stoppers
- 1 User Guide.

Please make sure that all products are included in your package and that all products are marked with respective article number. Make sure nothing is damaged or broken.

4. Explanation of the included products

To succeed with the installation process, it is imperatively important that you get familiar with the included products.

Harness with Transmitter and surcingle (Art. no 656SHA)

Repeater. (Art. no 653RLH)

Receiver (Art. no 651RXH)


Antenna Cable. (Art. no 358KHA)

Antenna. (Art nr 350AHA)

5. horseAlarm - How It Works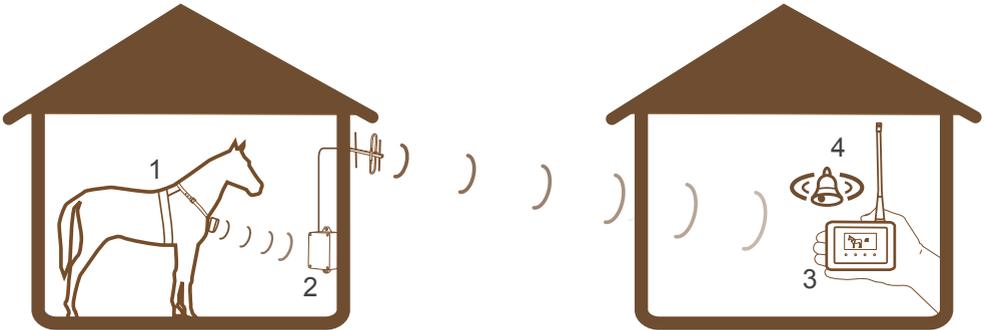

1. The sensors measure how much the horse is sweating and whether is standing or lying down.
2. The information is sent from the sensors to the Repeater, which retransmits the signal through the R300-antenna.
3. The receiver receives the signal wirelessly and the values as well as recent history are displayed on the LCD screen.
4. When the sensors on the horse register signs of foaling or colic, horseAlarm will alarm using both sound and visual indicators.

The harness with the Transmitter is equipped with three sensors. Two are placed on the inside of the harness and measures how much the horse is sweating. The third is placed in the Transmitter and feels when the mare is lying down flat on the side. The information is sent wirelessly to the Repeater.

Repeater can receive information from up to eight Transmitters at the same time. The information is retransmitted from the stable to the house using the R300 antenna. In the house the signal is picked up by the receiver, which shows the information on the screen. In Advanced mode you can follow your horse up to six hours back in time.

horseAlarm alarms with both light and sound. horseAlarm has a default setting suitable for the average user, but you can easily customise the alarm to suit your horse and environment.

INFORMATION


The signals are sent out like waves in the air. The waves can penetrate most objects, but the signal will get weaker for every object it has to go through. Always try to get as much of a free line of sight from antenna to receiver as possible to get the best signal.

6. Before You Begin the Installation

Read through the instructions in this manual before you begin installing you horseAlarm package.

Start with charging the receiver as it will be used later in the installation process. The battery is only partly charged and will therefore only work for a short while if not charged.

1. Connect the power adapter (12V) to the power input of the receiver and then to a wall socket (220V).
2. Check that the charging indicator is on.

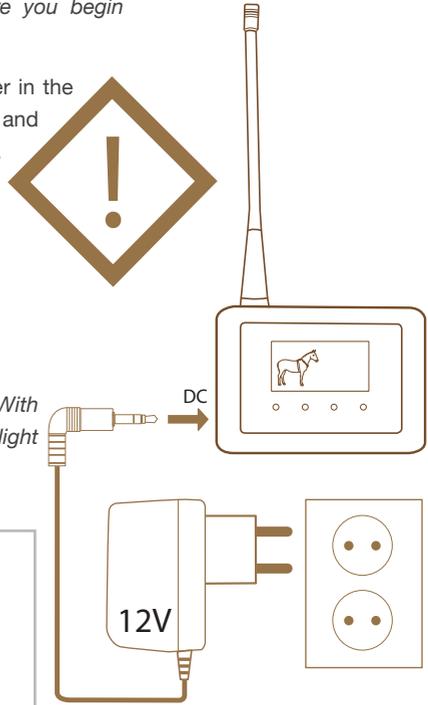A fully charged battery will last between 12 hours (With background light ON) to 30 hours (With background light set to AUTO)

NOTE!
Only use the 12V adapter marked *Receiver* to power the receiver! Usage of any other power source may damage the receiver and is NOT covered by the warranty.

3. Connect power to the repeater by using the power adapter marked with *Repeater/RadioLink only*. When you connect power to the repeater you will see a green flash.
4. Go to *Chapter 7 - Understanding and testing before use*.

7. Understanding and Testing Before Use

It is important that you are familiar and comfortable with the alarm and its functions before installing and start of use. The following chapter will take you through a demonstration of all important functions.

1. Turn on the receiver by pressing the ON-button on the top of the unit:

2. Make sure you are using the setting *Simple Mode* on the receiver by pressing the D-button until the screen below appears.

After about 40 seconds the *LINK indicator* will change color to red and the receiver will sound, telling you that the receiver is not receiving values from the transmitter.

3. Take the transmitter out of the harness pocket.

Unscrew the 4 screws to open the transmitter. Remove the small piece of paper between one of the batteries or insert new ones. This will turn the transmitter on and allow it to start sending values to the receiver. Keep the transmitter outside the pocket lying down flat “on its back”, with the logo facing upwards.

During the first 4 min that the transmitter has been connected to power, it will be in *fast mode*, sending values every 2 seconds to simplify the demonstration of the functionality for a first time user. After 4 minutes the transmitter will change to *normal mode* and send every 20th second to save power and give the user 6 months of battery time.

LINK should now have changed from red to green and stopped beeping. This means that the communication works between the units and that you now can test the functions for sweating [S] and position [P].

4. Make sure the receiver is set to alarm on either [S] or [P] by pressing the button (B) until the symbol [S/P] is shown in the upper part of the display. If you keep pressing the button (B) for a few seconds you will also change which position mode the receiver is set to: A, B* or C. Choose position *mode C* for *colic mode* which is the quickest alarm mode, the easiest to demonstrate.

*Not available in all countries.

5. Place the transmitter according to the picture below, to simulate that the horse is lying down:

Watch how the horse lies down on the display. Shortly the position alarm will go on in the receiver, shown by the red light and the beeping.

To turn off the alarm, turn the transmitter upright (with the logo up) and press the (C) button. **For highest possible security the transmitter continues to send the alarm 3 times**, (in case the first alarm would not reach the receiver because of unpredicted interference). **This means that you will have to turn off the alarm 3 times by pressing button (C).**

6. **To easily demonstrate the sweating alarm**, set the sweating level to a low level (such as 2 or 3), by pressing the (A) button until the number in the circle on the right side of the display is 2 or 3.

After you have set the sweating alarm level on the receiver, grip one of the sweating sensors as in the picture below:

When the measured sweat level is as high as your alarm level (or higher), the receiver starts to alarm with red light and beeping. To turn off the alarm, let go of the sensors and press the (C) button.


Advanced Mode:

Advanced Mode, (which also can be called *Graph Mode* or *History Mode*), is entered by pressing the (D) button and tells the user how the horse has been during the last 6 hours.

When used to demonstrate its functionality, it is recommended to reset the transmitter by taking out one of the batteries from the transmitter and insert it again. This will make the transmitter send more rapidly to simplify the demonstration.

In demonstration mode the whole display is filled with diagram information in 10 minutes. When used as normal, (4 min after it has been reset), it takes 6 hours to fill the display with graphs of the horse's behavior.

Lay the transmitter down on various sides and simultaneously grip the sweating sensors to create diagrams similar to the ones below: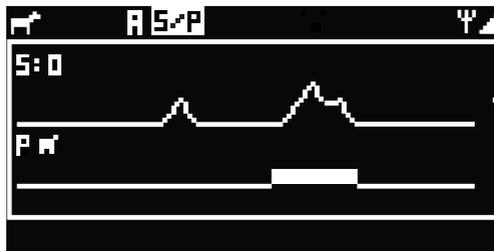

The alarms work in the same way as in *Simple Mode*.

The next step is to start the installation, please go to chapter 8.

8. Placement of harness, Transmitter & Repeater


The included cable will allow you to place the Repeater within 7 meters from the R300 antenna and within 2 meters* from a 220V socket. Maximum distance between horse and Repeater is 50m.

1. Check the state of the harness and sensors, clean the sensors and condition the leather if needed.
2. Open the Transmitter by unscrewing the four screws. Be careful not to stretch the cables!
3. Put the batteries in the Transmitter, close, screw back the screws, then put it back in the Transmitter pocket. Put the harness on the horse according to the illustration. To make it comfortable for the horse, close the surcingle at the top. Also make sure that the rubber stop is still on. It prevents the horse from opening the surcingle.
4. Mount the Repeater with the power and antenna connections facing down, within 7 meters from where you will place the antenna, and 2 meters from a 220V socket.
5. Screw the small black antenna onto the left antenna connection of the Repeater.
6. Connect the 12V adapter marked with *Repeater* to the Repeater and then to the wall socket. Make sure the Repeater is receiving a signal from the Transmitter, it should flash in green every 20th second.

INFORMATION


*To lengthen the power cable, use a regular 220V extending cable. If you due to security reasons do not wish to use high voltage cables close to your horses, you can purchase a low voltage extending cable from your retailer.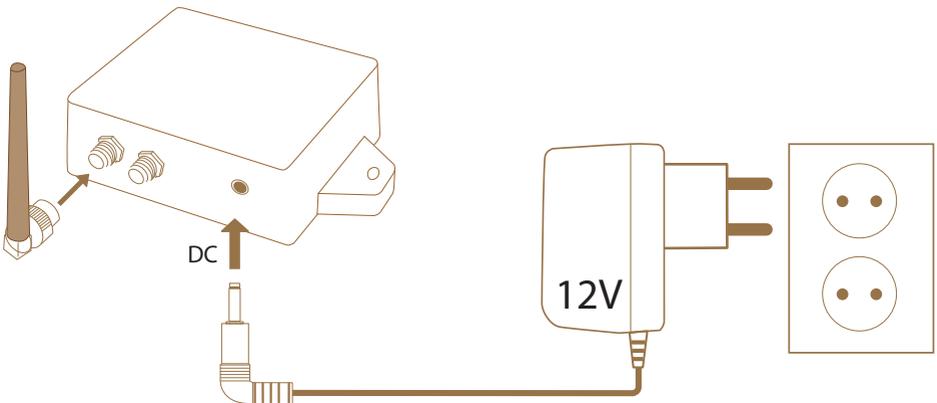

9. Mounting the Antenna

To achieve the best signal reception, mount the antenna facing the house you will be receiving the signal in, with no objects between.

1. Unplug the power to the Repeater, connect the antenna cable.
2. Drag the antenna cable to the point on the outer wall where you have chosen to mount the antenna.
3. Mount the R300 antenna temporarily with 1 or 2 screws. Put the shrinking tube on the antenna cable and connect it to the antenna. Plug in the power to the Repeater again.
4. Check the signal quality using the receiver at the intended receiving point. The scale to the upper right on the screen shows the signal strength. Adjust the antenna if needed, or try vertical position.
5. Mount the antenna with all four screws and heat the shrink tube over the antenna connection when you are satisfied with the signal.

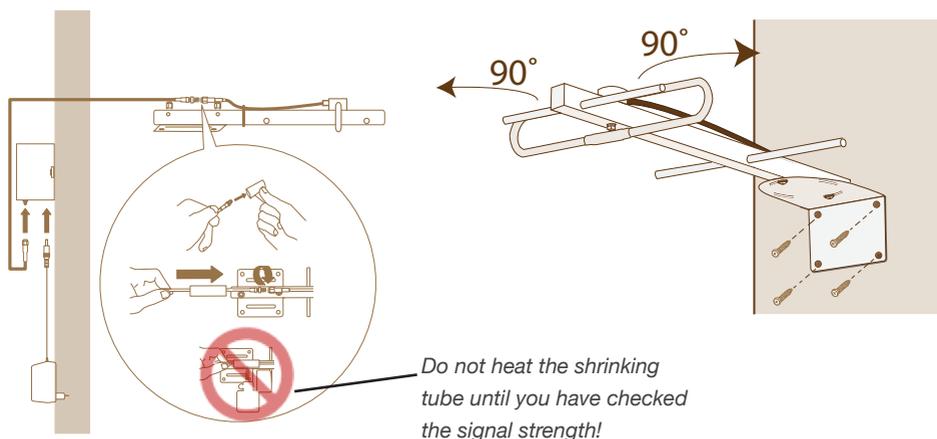INFORMATION

Be careful not to break the cable when going around corners etc. Especially in cold weather as the cable will be more sensitive.

Make sure there are no metal objects within 1 m of the antenna, as they may cause interference.

10. Receiver Mode - Easy

Easy mode is suitable for most users with only one horse, who are not interested in statistics and want an easy-to-understand display. We recommend that you do not change any settings to start with. horseAlarm will alarm if the horse is sweating or lying down for more than 7 seconds.

INFORMATION


If the sweating alarm is too sensitive, press the A-button to increase the alarm limit. After reaching the maximum value of 15 it will automatically go to 1.

If your mare is lying down flat on her side even though she's not foaling, or if it is very hot in the stable (the mare is sweating because of heat, not foaling) you may want to change the Alarm mode. Read more about this in the next chapter.

How to read the display:

If the horse is standing up, the horse in the display stands up as well. If the horse lies down flat on the side, the horse in the display will do the same.


To the right you can see the current sweat level of the horse and the sweating alarm limit.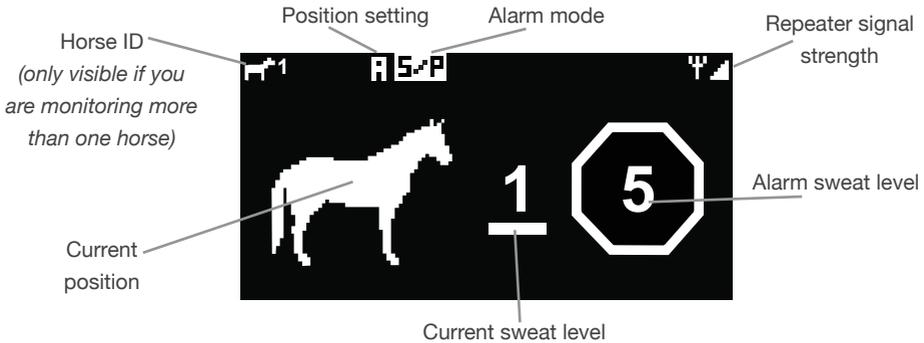

11. Battery symbols

On the receiver you can see the battery status of both the receiver itself and the status of the transmitters in your harness. The battery symbols look the same in both advanced and easy mode.

The battery symbol in the bottom left corner, represents the the harness and will show up when the battery level is low. You will see the *Horse ID* of the harness in question next to the battery symbol. The battery symbol for the receiver will show up in the top right when the battery is low. When you push a button the alarm is turned off and the symbol will go away.

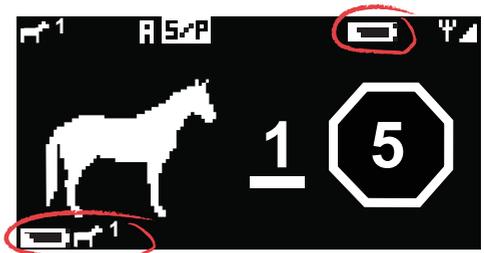12. Alarm Modes

All horses are different and sometimes you will need to change the alarm function on horseAlarm. For example, in hot condition you may need to rely on the position alarm only.

Press the B button to switch between the following modes:

Alarm off The receiver will not blink or sound at all.	S Sweat Will alarm when the horse is sweating more than the set limit. <i>Read the S paragraph.</i>	P Position Will alarm if the horse is lying down for a certain time. <i>Read the P paragraph.</i>	S+P Sweat or position The default setting. <i>Read both the S and P paragraphs.</i>	S+P Sweat & Position Will need both sweating and position to exceed their levels to alarm. <i>Read the S and P paragraph.</i>
--	---	---	--	--

S Sweat Settings

Press the A button to increase the alarm limit for sweating. After reaching the maximum level of 15 it will jump down to 1.

P Position Settings

Normally the mare is not sleeping flat on her side at the end of the pregnancy. Thus we can detect when she is lying down flat and assume that the foaling has begun. For normal foaling cases we recommend that you use setting A.

Press the B button for a few seconds to switch between the different position settings. Release when your chosen setting is visible on the screen. You can choose between the following:

A – The position alarm will alarm if the horse is laying down for more than 7 seconds. A contraction lasts for at least 7 seconds. If the horse is lying down for less than 7 seconds the alarm will not go off.

B – This setting is for mares resting flat on the side. For the alarm to trigger the mare has to lie down and get up again within 3 minutes. If she is lying down for longer than 3 minutes it is assumed that she is resting, not foaling, and the alarm is reset and will not go off when she gets up from the rest.*

C – For horses with colic or that are ill in other ways. The alarm will trigger immediately (2 sec) after the horse lies down. *Not available in all countries.

INFORMATION

To save battery life, the information is only transmitted every 20th second. The alarm will still alarm according to the descriptions for A, B and C above, as data is recorded every two seconds. The data recorded during the last 20 seconds is transmitted as a "package" and then interpreted by the receiver.

13. Receiver Mode - Advanced

In advanced mode the horseAlarm will keep track and draw the history of the values. This means that you can see what has happened during the last 6 hours, even if you have not been awake the whole time. Both sweating and position is recorded. The screen updates every three minutes. You can also see the stable temperature, making it easier to determine if the mare is sweating because of heat or foaling.

How to read the position diagram:


If the horse is lying on its right side, a thick line will appear **below** the thin line.

If the horse is lying on its left side, a thick line will appear **above** the thin line.

How to read the sweat diagram:

The diagram will display how much the horse is sweating, 0 is the least and 15 is the most.

The chosen limit is shown as a small dot at the right of the screen and also as a number. Adjust the sweat-limit with the A button. After 15 it will automatically go to 1.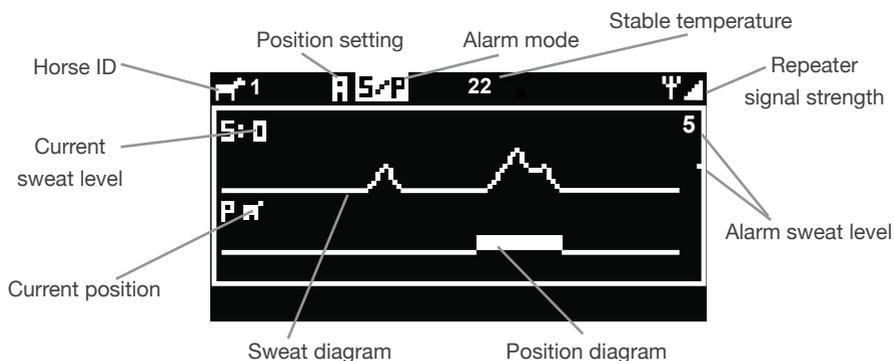

14. Receiver Mode - Settings

Press the D button for a few seconds until the settings screen appear.

Setting the background light

Press the A button to change the background light setting between ON and AUT.

The default setting is Auto (AUT), where the background will light up for five seconds after a button is pressed. Keeping the light on will shorten the battery life of the receiver.

Thermometer settings

You can change between Celcius and Fahrenheit by holding the B button down.

Here you also have the possibility to calibrate the temperature as it is sometimes offset during production. Press the B button to do this. Pressing one time increases the number with one degree.


Sound settings

Press the C button to change between OFF (0) or ON (1,2 or 3).

15. If you have more than one horse

Extra harnesses can be purchased from your retailer. The harness kit include both harness and transmitter. Extra harnesses are preset to ID 2.


Change the ID of the transmitter by opening the transmitter and setting the three switches in positions according to the picture below. Each transmitter need its own ID, which means you can monitor up to eight horses simultaneously as there are eight different IDs to choose from. How to set the switches for each ID is illustrated below.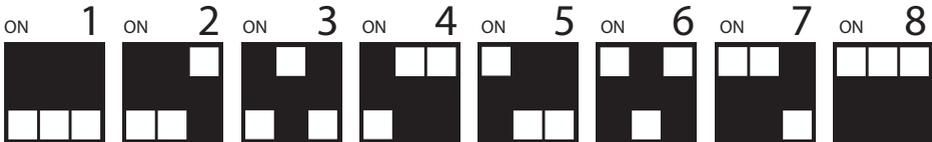

How to read the display with several horses

Let's say you are monitoring three horses at the same time. (ID 1, 2 and 3) You are currently looking at horse no 1, but horse no 3 is sweating more than the set alarm limit. Even though you are looking at horse no 1, the receiver will alarm as usual. You will see an indication in the lower right corner of the screen of which horse that has triggered the alarm.

Below is an illustration of what the indication looks like in Easy and Advanced mode respectively:

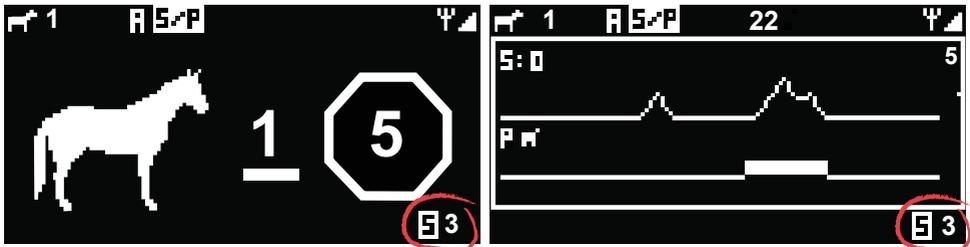16. Frequently Asked Questions

Is it possible to get alarm notifications sent to a mobile phone?

Yes, by using our GSM Alarmer accessory.

How many horses can I monitor simultaneously?

You can monitor up to 8 horses.

Can I adjust the alarm sound level?

Yes, please see chapter 14.

The alarm shows that the horse is lying down even though it just got up again?

The information is sent to the Receiver every 20th second. Thus, there is a slight delay before the position change shows up on the display.

Further questions?

Visit our website www.ludaelektronik.com or contact us in another way (see chapter 20, Contact us).

17. Troubleshooting

Please follow the steps below to locate your problem.

You can contact our support team at: <http://ludaelektronik.com/support/en/>

1. Is the Receiver working?

- If you can see a picture in the display the monitor is considered working. Please go to section 2.
- If you cannot see anything in the display but the ON LED is green, try pressing any button. The display should lit up.
- If the ON LED is not glowing, plug in the power adapter. Wait an hour in case the battery level is very low. If nothing happens, please contact the LUDA customer support.

2. Is the Link LED on the Receiver red?

- If it is, go to section 3.
- If it is green, please contact the LUDA customer support.

3. Is the Repeater working?

Disconnect the power from the Repeater and then connect it again. You should see a green flash when the power is plugged in. Within 1 minute you should see a second flash.

- If you didn't see any flash, please check that there is power in your wall socket. If you have power, contact the LUDA customer support.
- If you did see a flash at the start and then a second flash, please go to section 5.
- If you saw a first flash and then nothing or a constant green light, please go to section 4.

4. Is the Transmitter working?

Check if there is a green LED flashing every 20th second on the Transmitter box.

- If you didn't see any flash, change batteries in the Transmitter. If changing batteries doesn't help, please contact the LUDA customer support.
- If you did see a flash every 20th second, please go to section 5.

5. Check the distance between the Transmitter and Repeater

Try moving the Transmitter and Repeater closer to each other and see if you get a signal. If you get a signal you will see a green flash on the Repeater. Also check that the antennas are connected to the right connections. (See chapter 8.)

6. Check the distance between the Repeater and Receiver

Try moving the Receiver closer to the outdoor R300 antenna and see if you get a signal. Also check that the antennas are connected to the right connections on the Repeater. (See chapter 8.) Also make sure the antenna cable isn't damaged in any way.

18. Warranty

The warranty is due, if no other terms agreed, 12 months from the date the product is purchased from the authorised LUDA Elektronik AB dealer. During the warranty time LUDA Elektronik AB is obliged to repair, alternately, replace the broken product to a similar product or a product with better functions. When LUDA Elektronik AB replaces the product, the new product will be covered by the original warranty during the remaining warranty time. If your product is replaced, previously used, completely functioning parts might be used. Replaced parts are owned and controlled by LUDA Elektronik AB. The warranty does not cover damage that have arisen due to physical damage, accidents, natural disasters, the usage of incorrect power adaptors or usage of the products in deviant behaviours from what is recommended by LUDA Elektronik AB and in this user guide. The end consumer will be responsible for all shipping costs that occur from transportation to the LUDA Elektronik AB repair shop. The warranty will only be valid if the original receipt, issued by the authorised LUDA dealer, can be presented. The original receipt needs to contain information regarding place of purchase, issue date and product description. LUDA Elektronik AB reserve the right to refuse to perform the warranty service if above mentioned receipt information has been removed, changed or is missing.

EN

ES

FI

IT

PL

SV

19. Information regarding recycling

The crossed-out wheeled-bin symbol on your product, battery, literature, or packaging reminds you that all electrical and electronic products, batteries, and accumulators must be taken to separate collection at the end of their working life. This requirement applies in the European Union. Do not dispose of these products as unsorted municipal waste.

Always return your used electronic products, batteries, and packaging materials to a dedicated collection point. This way you help prevent uncontrolled waste disposal and promote the recycling of materials. Batteries can contain mercury, lead or cadmium - substances that are harmful to our health and our environment. More detailed information is available from the product retailer, local waste authorities or your national producer responsibility organization.

20. Contact us, accessories and support

LUDA Elektronik, founded in 2003, is a leading distributor of wireless video equipment. We have in-depth expertise on wireless video technology and its usage areas. Thanks to this we can offer our customers the latest products and high quality sales support, service and customer support.

Accessories and support

Visit www.ludaelektronik.com/support/en/ for support and accessories to your horseAlarm.

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Tel: +46 (0)31 3130290

Fax: +46 (0)31 3130299

E-post: service@horsealarm-europe.com

Declaration of Conformity

R&TTE Declaration of Conformity (DoC)

Unique identification of this DoC: 650HAL-2010-CE

We, LUDA Elektronik AB,

Aschebergsgatan 46

411 33 Göteborg

Sweden

Phone: +46 31 3131290

declare that the product:

product name: horseAlarm

type or model: 650HAL

to which this declaration relates is in conformity with

the essential requirements and other relevant requirements of the R&TTE Directive (1999/5/EC).

The product is in conformity with the following standards and/or other normative documents:

HEALTH & SAFETY (Art. 3(1)(a)):

EN 50371

EMC (Art. 3(1)(b)): Electromagnetic Compatibility (EMC) Directive 2004/108/EC

EN 301 489-1/-3

SPECTRUM (Art. 3(2)):

EN 300 220

Technical file held by:

LUDA Elektronik AB

Aschebergsgatan 46

411 33 Göteborg

Sweden

Place and date of issue (of this DoC):

Göteborg, 2010-01-01

Signed by or for the manufacturer:

Name (in print): Ludvig Brost

Title: CEO

Signed by or for the manufacturer:

Name (in print): Daniel Ludwiszewski

Title: Product Manager

R&TTE Declaration of Conformity (DoC)

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Vi, LUDA Elektronik AB,